



Drug Stability Data

CareVis Drug Stability Data Table 1/2

Medication	Diluent	Concentration	Storage Frozen (~ -20 °C)	Refrigerated (~ 4 °C)	Room Temp. (~ 25 °C)
5-Fluorouracil	NS	50 mg/ml	-	-	7, 21, 30, 84 d
5-Fluoruroacil	NS	50 mg/ml	-	-	28 d at 25°C + 7 d at 32°C
5-Fluoruroacil	NS	5 mg/ml + 50 mg/ml	-	-	21 d
Acyclovir	NS	5 mg/ml	-	-	8 d
Amikacin	NS	10 mg/ml	-	-	3 d
Bupivacaine	NS	1.3 mg/ml	-	-	30 d
Bupivacaine Hydrochloride	NS	7.5 mg/ml	-	84 d	7, 21, 30 d
Bupivacaine Hydrochloride + Sufentanil	NS	6.75 mg/ml + 5.0 µg/ml	-	84 d	7, 21, 30 d
Ceftazidime	NS	6 mg/ml + 48 mg/ml	-	-	8 h
Ceftazidime	NS	3.75 mg/ml + 32 mg/ml	-	7 d	8 h
Ceftazidime	NS	16 mg/ml	-	-	24 h
Cefuroxime	NS	20 mg/ml	-	24 h	-
Ciprofloxacin	NS	0.5 mg/ml +	-	-	24 h
Ciprofloxacin	NS	2.0 mg/ml	-	-	24 h
Ciprofloxacin	NS	0.5 mg/ml + 2.0 mg/ml	-	14 d	-
Clindamycin	NS	6 mg/ml	-	-	72 h
Cyclophosphamide	NS	10 mg/ml	-	-	3 d
Dacarbazine	NS	2 mg/ml	-	24 h	-
Deferoxamine	NS	200 mg/ml	-	-	7 d
Deferoxamine	NS	100 mg/ml	-	-	10 d
Deferoxamine	NS	5 mg/ml	-	-	28 d at 25°C + 7 d at 32°C.
Deferoxamine Mesylate	NS	10 mg/ml	-	-	7, 21, 30 d
Deferoxamine Mesylate	NS	100 mg/ml	-	-	7, 21, 30 d
Doxorubicin Chlorohydrate	NS	2 mg/ml	-	-	14 d
Doxorubicin Chlorohydrate + Vincristine Sulphate	NS	330 µg/ml + 11.5 µg/ml	-	-	48 h
Epirubicin	NS	0.3 mg/ml	-	-	10 d
Flucloxacillin	NS	20 mg/ml	10 d	-	-
Flucloxacillin	NS	50 mg/ml	10 d	-	-
Flucloxacillin	WFI	20 mg/ml	10 d	-	-
Flucloxacillin	WFI	50 mg/ml	10 d	-	-

NS: Normal Saline 0.9%, D5W: Dextrose Solution 5%, WFI: Water for Injection


CareVis Drug Stability Data Table 2/2

Fluconazole	NS	1.8 mg/ml	-	-	72 h
Ganciclovir	NS	5 mg/ml	-	-	4 d
Gentamycin Sulphate	NS	1.8 mg/ml	-	-	3 d
Leucovorin	NS	1 mg/ml	-	-	3 d
Levobupivacaine	NS	1.25 mg/ml	-	-	7, 30 d
Meropenem	NS	5 mg/ml + 40 mg/ml	-	-	8 h
Meropenem	NS	3 mg/ml + 7.5 mg/ml	-	3 d at 5°C	8 h
Mesna	NS	32 mg/ml	-	7 d	1, 2, 7 d
Mitoxantrone	NS	0.6 mg/ml	-	-	9 d
Morphine Hydrochloride	NS	1 mg/ml	-	84 d	7, 21, 30 d
Morphine Sulphate	NS	1 mg/ml	-	84 d	7, 21, 30 d
Penicillin G-K	D5W	10,000 units/ml	30 d	24 h	24 h
Penicillin G-K	D5W	100,000 units/ml	30 d	10 d	24 h
Penicillin G-K	NS	100,000 units/ml	30 d	10 d	24 h
Piperacillin	NS	28 mg/ml + 150 mg/ml	-	-	24 h
Piperacillin	NS	19 mg/ml	-	14 d	24 h
Piperacillin	NS	7.5 mg/ml + 50 mg/ml	-	14 d	24 h
Ropivacaine	NS	1.8 mg/ml	-	-	30 d
Ropivacaine Hydrochloride	NS	7.5 mg/ml	-	84 d	7, 21, 30 d
Trabectedin	NS	0.01 mg/mL			48 h
Vancomycin Hydrochloride	NS	10 mg/ml	-	-	3 d
Vancomycin Hydrochloride	D5W	20 mg/ml	30 d	10 d	24 h
Vancomycin Hydrochloride	NS	20 mg/ml	30 d	10 d	24 h
Vinblastine	NS	0.3 mg/ml	-	-	9 d

NS: Normal Saline 0.9%, D5W: Dextrose Solution 5%, WFI: Water for Injection

Contact Information
Exclusive Distributer for CareVis® Single Use Infusion Pumps


PROMECON GmbH

Kreuslerstr. 10 | 20095 Hamburg | Germany



Tel.: +49 40 3690 1690 | Fax: +49 40 3690 1699

info@promecon-medical.de | www.promecon-medical.com

DIN EN ISO 13485:2016



S&S Med CO., Ltd.

67 Bongseong-ro, Gunpo-si, Gyeonggi-do, 15848,
Rep. of South Korea

Disclaimer

The information supplied has been based upon the current level of information available, for the purpose of specifying the requirements regarding environment, health, and safety in conjunction with the product. They are not to be interpreted as a warranty for specific product characteristics.