Promecon GmbH
 Kreuzerstrasse 10, D-20095 Hamburg, Tel.: +49 40 36 90 16 90, Fax: +49 40 36 90 16 99
 Email: info@promecon-medical.de, Url: www.promecon-medical.com

EN **Instructions for Use**
OptiGuard QuickChange

Camera drape concentrically wound and with optic adapter, closed system

Important

1. This product was designed, tested and manufactured for single patient use only.
2. Any attempt to reuse or reprocessing may lead to failure and potential patient injury.
3. Reprocessing and/or resterilisation of this product may give rise to patient infection through contamination.
4. Do not reuse, reprocess or resterilise this product!

Indication

The camera drape is a sterile, single use, disposable drape made of tough, thin polyethylene, designed for surgical procedures. It is the quickest and simplest way to create a sterile field around a video system during operative laparoscopic procedure.

Directions for use:

1. Sterile person holds the ring on the outside and pushes the optics adapter slightly outside of the ring, so that the camera with the camera cable can be attached by the non-sterile person (picture a). Check correct positioning!
2. The non-sterile person now pushes the optics adapter with the camera through the ring, so that the sterile person can grasp it.
3. While the sterile person holds the optics adapter, the non-sterile person pulls the sleeve ring slowly and steadily over the complete cable, being careful not to touch the surface of the drape.
4. Sterile person opens bayonet, introduces the endoscope, and closes bayonet. Check correct positioning!
5. Blue sealing tape can be used to seal the endoscope and then the sleeve (picture b).

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